**WARRANTY SHEET /
GARANTIESCHEIN**

Final inspection:

Name:

Signature:

Identification tag / Product type (write up):

EN

DE

Warranty conditions:

1. On this product is applied 24 months warranty from the date of sale. The warranty is applied for material and manufacturing defects.
2. The product has to be used entirely for the purpose it is designed for and the instructions in the user guide must be followed.
3. **The product has to be properly used, maintained, cleaned and stored in accordance to the user guide instructions.**
4. Do not pull down the identification tag. It has to be readable and undamaged for the whole warranty period.
5. The warranty case has to be claimed by the seller, where the product was bought. Properly filled "Warranty sheet" and the proof of purchase must be declared by the warranty application. Every defect should be claimed immediately
6. The claim procedure can be started when the device is delivered complete, in original packaging, carefully packed and secured against damage. The owner is committed to deliver the device clean in accordance to the general hygienic measures. The basic maintenance and cleaning cannot be expected or even required when claimed.
7. The warranty does not cover: failures caused by the overloading of the products, accident, forced damage, unauthorized changes made on the product and/or its accessories, natural wear and tear of parts, especially upholstery dirt and damage, wear of plastic parts, brakes, tyre profile and tyre puncture, etc.
8. The warranty is terminated whenever the following conditions arise: the user fails to follow the instruction mentioned in this user guide and manufacturer's storage and maintenance suggested procedures, or whenever the product is serviced by any other party except of authorised service centres, or when the identification label is not stucked on the stroller and also in the warranty sheet.
9. The warranty period cannot be interchanged with using period, that means for the lifetime period when correct use and maintenance of the product in accordance to the user guide are fulfilled.
10. The producer nor the importer is not liable for any losses and injuries caused by manipulation and usage of the product in conflict with the "User Guide" instructions.

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**TOM 5 Streeeter-SE
TOM 5 Streeeter
(sizes: MINI, STD, MAXI)**

Date of sale (day, month, year): _____ Name (block letters), phone number, stamp and signature of the seller: _____

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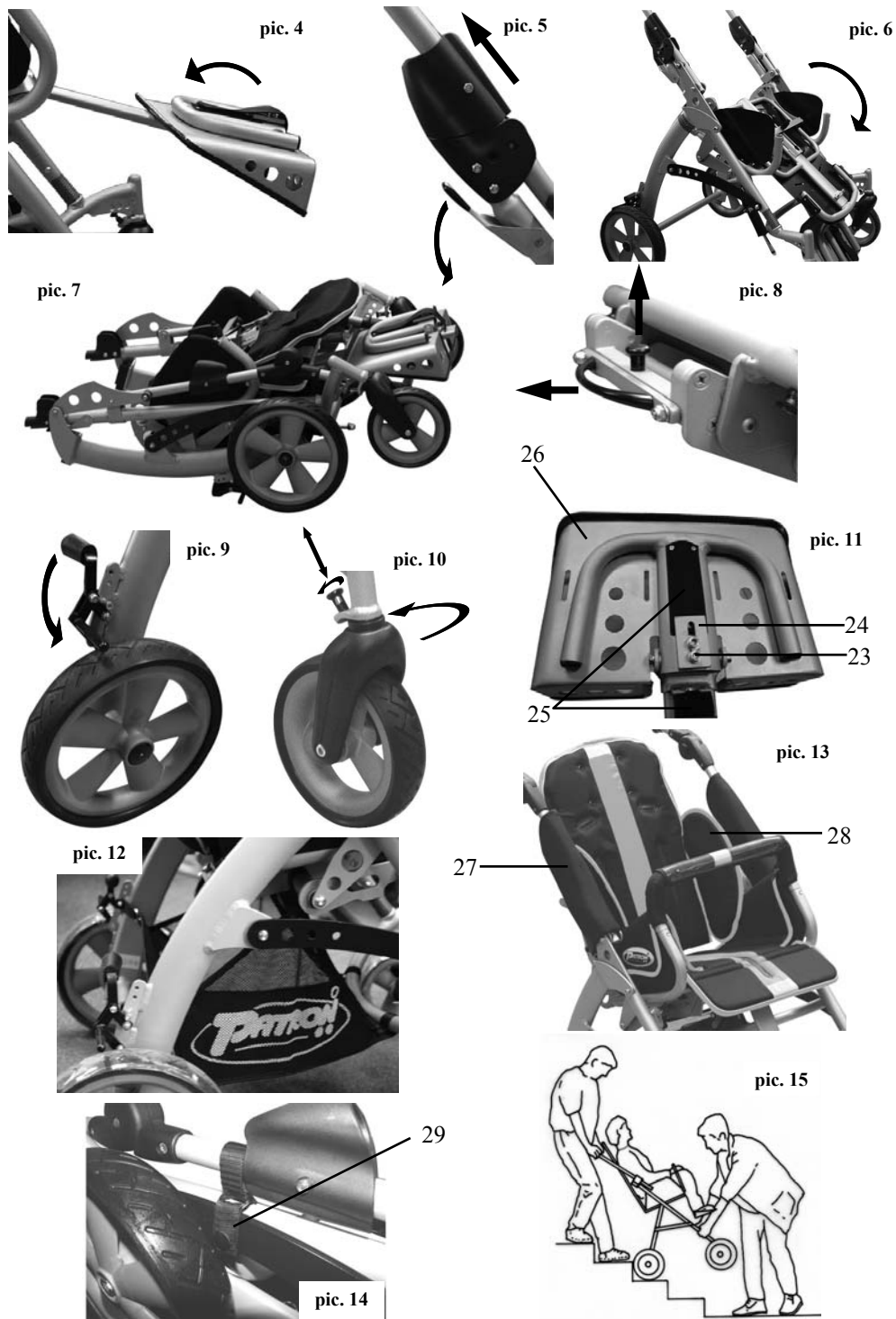
multipurpose fully foldable aluminum stroller of new generation - TOM 5 Streeter - is designed for both short and long term stay of children with special needs. Brand new full-aluminum chassis of Tom 5 Streeter is produced by new high-tech ALUX production technology. Stroller excels in the high-tech s design and high durable construction of seat and chassis. Products is ranked among the lightest strollers in its category. TOM 5 Streeter's unique design of the seat features enables the user to apply enhanced range of special fixing accessories. Stroller's size range is determined for occupants with a very wide variety and levels of disabilities and requirements. In case you are not fully sure about the usage of the product while reading user guide, please contact your local official distributor for advise. Enjoy life to the full with TOM 5 Streeter !

..... SAFETY AND WARNING NOTES:

① **CAUTION! Any load attached to the handle (bag, ...) affects the stability of the stroller!**

② **The side padded frame covers must be used properly all the time!**

③ **Recommended tire air-pressure: 230 / 230 kPa**

⑥ **Footrest velcro lock must be properly tightened while using stroller!**

.....maximum load.

.....weight of the product

WARNING! Keep fingers out of any moving parts when operating with the stroller.

WARNING! Never leave your child unattended! Even if the belts and brakes are in use.

WARNING! Check before use that all locking devices are properly locked and all safety equipments are properly used.

WARNING! Check before use that seat unit is properly locked in the chassis interface.

WARNING! The seat unit is not suitable for a child younger 6 months.

WARNING! This product must not be used while jogging or in-line skating.

WARNING! The seat unit must not be used in motor vehicles as the car seat.

ATTENTION! Read this manual carefully before the first use of the stroller, as well as keep it for further reference. Safety of a child might be in danger in case you will not follow the instructions and safety notices mentioned I the user guide.

- The attendant must be familiar with the instructions of this manual.
- Professional medical advice regarding the suitability of this product must be sought before use.
- Always make sure, before departure, the child is comfortable, and that clothes and rugs etc, are tucked in, so they do not fall in the wheels.
- The stroller MUST NOT be used as a seat for a child while traveling in motor vehicle or bus.
- Ensure that child is clear of moving parts before making adjustments, to avoid any injury.
- When angle-positioning and adjusting the seat and backrest act accordingly to avoid sudden uncontrolled movements.
- Change of the seat direction must be done without the occupant and with empty seat unit.
- Do not allow the child to climb unassisted into and out from the stroller.
- Always apply the parking brakes, when getting in and out the stroller.
- Ensure yourself that the fixing accessories, safety bar and other equipment is fitted and adjusted correctly.
- Do not stand on the footrest.
- The brakes must be engaged at all times when the stroller is not moving.

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NÁVOD NA POUŽITIE

- Getting over the curb practice firstly with an empty stroller.
- The attendant must take precautions against the fire risk. In buildings, check the location of fire exits and ensure procedures are understood.
- Do not let children play with or hang on the stroller.
- During the drive keep full attention, to avoid any dangerous situation or collisions.
- Do not overload the stroller. Overloading of a chassis, seat, bag or basket causes reduction of stability of the stroller.
- Make the full attention, when manipulate with the stroller, not to injure the child or yourself.
- If it is possible, always use drive-up ramps or lift to overcome any access restrictions.
- Always lift the stroller in two persons.
- Never transport the stroller when you and/or assistance are not sure if you can transport the stroller with the child safely.
- When you drive the stroller on the road, follow the road traffic regulations.
- Do not use the stroller for any other purposes then it is designed for.
- The stroller is designed to carry only one child.
- Do not use any other than original PATRON accessories designed for this product.
- When the stroller is folded and unfolded, the user guide must be followed carefully, to avoid any damage on the construction what could affect the product stability.
- In case of apparent defect, which limits product's safe using (e.g. damage of chassis, part falls away or other optically visible defects, cracks etc.) it is necessary to stop using the product immediately and contact your local official distributor or service centre.
- Company PATRON does not bear any responsibility for the damages caused by using other than original PATRON accessories supplied by this company.
- Company PATRON does not bear any responsibility for the damages and accidents caused by inappropriate use of this product.

USER GUIDE

1) Chassis:

Assembling the stroller (with attached seat):

Place folded stroller on wheels. Unbutton and release the transport safety-catch on frame. To open the undercarriage stay between the rear wheels and hold handle tightly. Move handle up and backwards until the handle folding release is fully locked in its end position. To check the locking position, move upwards the handle to ensure that both mechanism are fully locked (the undercarriage must not start folding). Pull the seat tilt-in-space adjustment lever (pic.5) and move the seat to the requested position. Pull the padded frame covers downwards over the locking mechanism and close both covers by zippers properly. Release backrest recline adjustment lever and set the position of backrest. Tighten lever properly. Release the footrest angle adjustment lever, set the position of footrest and tighten lever again. Unfold the footplate and lock it by velcro fastener. Attach the safety bar. Attach and adjust fixation accessories if needed. The stroller is assembled.

Folding the stroller (with attached seat):

Apply the parking brake and take off the safety bar. Release the footrest angle adjustment lever, set the position of footrest a bit less than horizontally, ca. 20 degrees (pic.4). Release the footrest velcro lock and close the footplate (pic.4). Release the backrest recline adjustment lever and push backrest as far as possible forward, towards the seat base. (It is recommended to take out the exceeding fixing accessories and adjust the seat width by side plates to the maximum limit, which enables to decrease the size of the folded seat). Tighten the lever. Unzip the padded frame covers partially and pull over the locking mechanism upwards. Pull the seat tilt-in-space adjustment lever (pic.5) and set the position of the seat to tilt forward to maximum (pic.6). Lift the handle folding release movable parts (pic.6) on both sides and fold the handle forward towards the front wheels. Ensure folded undercarriage by transport safety-catch. The stroller is folded (pic.7).

Assembling and disassembling the undercarriage:

Folding - Apply the parking brake. Unzip the padded frame covers partly and pull over the locking mechanism upwards. Lift the handle folding release movable part (pic.5) and fold the handle forward towards the front wheels. Ensure folded undercarriage by transport safety-catch. The undercarriage is folded (pic.7).

Assembling - use the same procedure, but start from the end of instruction above.

Chassis-and-Seat Interface (CSI):

Taking off the seat:

Apply the brake. Release the interface lock safety pin (pic.9) and hold it. Simultaneously pull (pic.10) and unlock the seat interface locking release (pic.8). Lift the seat.

Fitting the seat in the driving direction

Place the seat exactly into frame seat fittings. FIRSTLY always start to insert just a side of the seat into the FREE interface socket (without locking pins). By pushing down the second side, the seat will be locked automatically (you will hear the locking sound). Finally check the seat proper locking by slight lifting of the seat unit. Seat must not separate from the interface frame.

Fitting the seat against the driving direction:

To place the seat against the driving direction follow the same procedure like in previous chapter except of turning the seat 180° to be facing backside to front.

Parking Brake:

To engage the brake, press brake lever pad on both sides fully till the end position (pic.9). To disengage the parking brake, lift both brake lever pads.

Swivel lock:

If needed, the front wheel forks can be locked against vertical rotation. To lock the castor forks lift and turn the swivel lock pin by 90° on both sides (pic.15) and let it to snap in in the designated hole on the castor fork (pic.10).

Handle height adjustment:

To change the handle height position press the black plastic buttons on the handle hinges on both sides simultaneously (pic.2), set the handle to desired position, release buttons. Handle will lock in new position automatically, you will hear the locking sound.

Safety Bar:

To take off the safety bar (pic.12) press pin release on both sides simultaneously and pull the bar upwards to remove it. To insert the safety bar, both pin releases have to be pointed ahead. Press slightly both press pins and insert the safety bar into the tube inlays at once until the pins are locked. Check the correct position of safety bar by slight pulling up, safety bar must not lift.

2) Seat-unit positioning and adjusting::

Seat-unit tilt-in-space positioning:

Unzip the right padded frame cover partially from bottom upwards to be able to touch the reclining lever (pic.5). Hold the seat backrest firmly by left hand. Pull by right hand the seat tilt-in-space lever and set the position of the seat (19, pic.2). Release the lever and close the zipper of padded frame cover fully till the lower end.

Seat depth adjustment:

Strip off the seat base upholstery from the seat base (attached by velcro fasteners). Loosen two hexagon-bolts placed under the frontal middle part of seat by wrench key (21, pic.2). Wrench key is placed in backrest rear pocket. Loosen by hand 4 plastic nuts placed under the seat (19, pic.2). Set the desired depth-size of the seat by pulling or pushing. Properly tighten all hexagon-bolts and plastic nuts and place back the seat base upholstery.

Seat-base width adjustment:

Take off the seat base upholstery from the side plates. Loosen by hand 4 adjusting plastic nuts placed under the seat (19, pic.2). Set the seat width by pulling or pushing of the side plates. Properly tighten all plastic nuts and cover back the side panels by upholstery.

Backrest recline positioning:

Hold the seat backrest firmly. Loose backrest recline adjustment lever (7, pic.1 and 3). Set the position of backrest. Tighten the lever properly. If needed unzip partially the zipper placed on the rear part of backrest cover to make the manipulation easier. Check the correct locking of the backrest position. Backrest must stay in preset position.

Backrest height adjustment:

Strip off the backrest upholstery from the rear and front side of the backrest (attached by velcro fasteners). Loosen two hexagon-bolts placed on top part of backrest rear side (5, pic.3). Loosen two plastic nuts on the rear side of backrest (6, pic.1 and 3). Set the height of the backrest by lifting or pressing down. Properly tighten all hexagon-bolts and plastic nuts and place back the backrest upholstery.

Footrest set angle positioning:

Hold the footrest firmly. Release the footrest angle adjustment lever (20, pic.2). Set the angle of footrest set. Tighten properly the lever to fix its position.

Footrest-set height positioning:

Loosen two hexagon-bolts placed on rear side of footrest slider profile (22, obr.1). Set desired position of footrest. Properly tighten both hexagon-bolts.

Footplate angle adjustment:

Loosen two hexagon-bolts placed under the footplates (23, pic.11). Set the angle of footplate by pulling or pushing of footplate angle adjustment slide-plate (24, pic.11), which is determine requested angle of the footplate. Properly tighten both hexagon-bolts.

Footplate velcro lock:

Tighten and fasten at any time the velcro lock fastener placed under the footplate (25, pic.11) to ensure fixed footrest position. (security device against folding of footplate).

3) Stroller in operation:

Getting into the stroller:

Engage parking brake. If necessary ask trustful person for help to hold the stroller firmly. Place the child into the stroller carefully. Attach the safety bar and adjust the footrest position. Pull on and adjust all fixation and positioning accessories the stroller is equipped with. Always make sure that the child is seated comfortably in the stroller.

Getting out of the stroller:

Engage parking brake. If necessary ask trustful person for help to hold the stroller firmly. If required to easy getting out of the stroller adjust footrest position. Take off all applied fixation and positioning accessories. Take off the safety bar and get the child carefully out of the stroller.

Getting over the curb stones and low obstacles:

Down - Drive the front wheels to the edge of the curb. Hold the handles firmly and tip the stroller backwards and balance on the rear wheels. Drive the rear wheels to the edge of the curb, lower the stroller down the curb slowly on its rear wheels.

Up - Drive the front wheels close to the curb. NEVER bump the front wheels into the curb stone!. Hold the handles firmly and tip the stroller backwards and balance on the rear wheels. Drive forward and place the front wheels on top of the kerb. Position the rear wheels at the curb tone bottom edge and lift to the upper level.

Manual transport of the stroller:

- make use of ramps and lifts when they are available

- if there is no alternative and steps should not be avoided at least one trustful assisting person must be available to give assistance

- do not attempt to take the stroller up or down over the number of steps unless you and your assisting person are not sure to have the full control during the manipulation as well as he or she can absorb the weight of its occupant and stroller.

- take the rests in regular intervals, if you are tired

- occupant hands must be inside the seat unit during the manual transport time

- NEVER lift the stroller holding the safety bar or footrest!

Going stairs downwards:

Tightly hold the handle and position the stroller squarely with the front wheels on the top step. Tip the handle backward and push the stroller forward until the rear wheels are at the end of top step. Assisting person stands in front, holding the stroller by gripping the front frame. Lift the stroller by both persons at once. Carry the stroller down the steps. Continue step by step to the bottom surface (pic.15).

Going stairs upwards:

Tightly hold the handle and position the stroller squarely with the rear wheels against the bottom step. Tip the handle backward and push the stroller forward until the rear wheels are at the end of bottom step. Assisting person stands in front of stroller, holding the stroller by gripping the front frame. Lift the stroller by both persons at once. Carry the stroller upstairs. Continue step by step to the top surface (pic.15).

Upholstery:

Seat upholstery is fully detachable and hand washable by lower temperature (max. 40°C).

Backrest cover – put on the cover top part over the upper part of backrest, cover the front part, slip the cover into the gap between the backrest and seat base. Secure the lower end of cover by velcro fastener at the back of the backrest.

Seat base cover – put on the covers of the seat base side panels one by one, place the cover in correct position onto the seat base and cover by overlapping parts the side panels, fix by dry zipper.

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4) Standard equipment

Padded frame cover:

Attaching – put on the frame sides and connect the zipper. Pull over the locking mechanism (left side). Pull over the locking mechanism and tilt adjusting lever (27, pic.13) (right side). Close both zippers fully till the upper end.

Detaching –unzip both zippers and take off padded frame covers (when folding the stroller just unzip the covers partly from top and then pull over the locking mechanism upwards).

Hip rest:

Attaching – mount the hip rest by plastic clip onto the vertical side tubes of the backrest (28, pic.13).

Detaching – fold the hip rest inside the backrest and release the plastic clip from the side tube.

Basket below the stroller :

Attaching – Attach the baskets centre part over the undercarriage middle cross bar using fasteners. Fix it with the poppers. Attach the baskets rear part, interlace the fasteners through the side loops placed on the rear frame profiles. Fix it with the poppers. Attach the baskets front part over the undercarriage front curved cross bar using fasteners. Fix it with the poppers (pic.12).

Detaching – unfasten and release all fasteners from the anchoring points.

Transport safety catch:

Attaching – pull the transport safety catch fastener around the connecting bar and front frame vertical tube. Interlace the fastener through the loop and fix it with the popper. (pic.14).

Detaching – unfasten and release the fastener.

5) Optional equipment:

This product can be optionally supplied with very wide range of fixation and positional accessory. Please find the optional accessory user guide as separate brochure.

MAINTENANCE

The stroller has been designed to the high standard and with correct usage and maintenance will provide many years of trouble free service.

- make sure that you are familiar with the operation of the stroller. It should always open and fold easily. If it does not, do not force the mechanism - stop and read the instructions
- regularly inspect handle folding release, safety bar fittings, locking devices, catches, footrest, joints fixings and handle release safety clip to ensure they are secure, clean and in full working order
- brakes, wheels, tyres and upholstery are subjects to wear and should be replaced if necessary by a service centre.
- If you have inflatable tires on the stroller, check its inflation regularly. Allowed tire unevenness on the perimeter of tire is +/- 2 mm
- if the stroller is damaged or in need of any repair, only authorised service centres are allowed to provided this services.
- only spare parts supplied by company PATRON has to be used for repairs.
- should parts of your stroller become stiff or difficult to operate, ensure that dirt is not the cause, apply a light application of a spray lubricant e. g. WD 40 (do not use oil or grease). Ensure that the lubricant penetrates between the components but do not overspray as this may encourage the accumulation of dirt. Wipe off any spray excess. Regular lubrication of moving parts will extend the life of your stroller and make opening and folding easier.
- it is recommend to use rain cover when it rains (it is not standard equipment of the strollers)
- do not expose the stroller to the long-acting intensive solar radiation
- textile components of stroller and/or accessory must be used and stored in the way that the transparent plastic parts are not folded or turned tightly. Those can be damaged if the temperature is 5°C below zero.
- wheels, plastic and metal parts may be sponged clean with warm water and mild detergent.
- the seat fabric must be hand washed only at maximum temperature of 40°C, do not use laundry dryer.
- never clean with abrasive, ammonia based, bleaches and alcohol based cleaning compounds.
- after exposure to damp conditions, dry the product off with a soft cloth and leave fully open in warm and dry environment.
- store your stroller in a dry, safe place with recommended temperature 5-20°C.
- it is recommended that the stroller is checked by an approved service centre every 12 month

NOTES

PATRON Bohemia a.s. reserves the right to make design changes to any of their products without prior notice as a part of their continuous development program.

SCHEMA DESCRIPTION

- | | |
|---|--|
| 1. Handle | 17. parking brake |
| 2. Handle height adjustment hinge | 18. Basket brackets |
| 3. Handle folding release | 19. Seat width adjustment plastic nuts |
| 4. Backrest | 20. Footrest angle adjustment lever |
| 5. Backrest height adjustment hexagon-bolts | 21. Seat depth adjustment hexagon-bolts |
| 6. Backrest height adjustment plastic nuts | 22. Footrest height adjustment hexagon-bolts |
| 7. Backrest recline adjustment lever | 23. Footplate angle adjustment hexagon-bolts |
| 8. Hood bracket | 24. Footplate angle adjustment slide-plate |
| 9. Interface lock safety pin | 25. Footplate velcro lock |
| 10. Interface locking release | 26. Footplate |
| 11. Tilt-in-space adjustment lever | 27. Padded frame cover |
| 12. Safety bar | 28. Hiprest |
| 13. Footrest | 29. Transport safety catch |
| 14. Front swivel castor wheel | |
| 15. Swivel lock | |
| 16. Rear wheel | |

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- Alternativ steht Ihnen PATRON Deutschland GmbH für Produktinformationen zur Verfügung.
- Es sollen nur Ersatzteile PATRON Deutschland GmbH verwendet werden.
Technische Änderungen an den Produkten der PATRON Deutschland GmbH sind vorbehalten.